




**Proper Assembly of VixOne Nebulizer.** Proper assembly will assure that the VixOne nebulizer functions optimally and does not leak. Misassembly may cause leaking, diminished aerosol output and other issues.

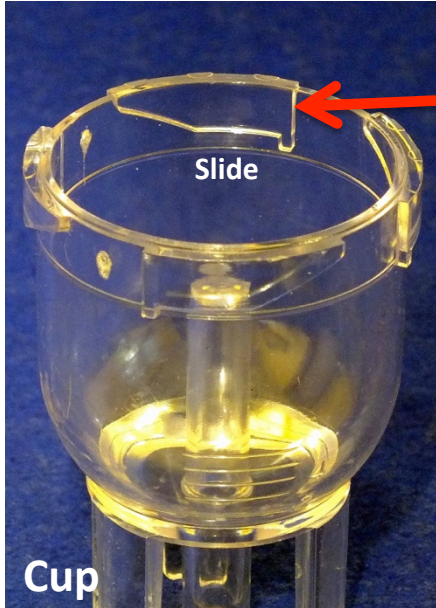

The VixOne cup has 4 'bayonet' **slides** located around the top edge facing out at 90° angles from each other.

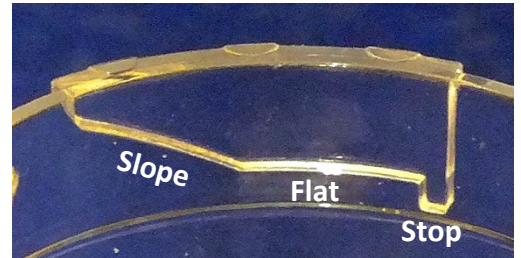

The shape of the slides allows matching surfaces in the cap to engage **tabs** on the cap that will ride on the **sloped** part when the cap is rotated onto the bowl.

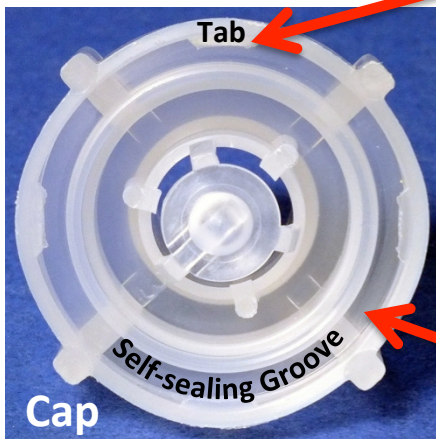

The tabs will then run along the **flat** surface briefly before encountering the **stop**.

The cup also has a **top edge** that is designed to fit snugly into a **self-sealing groove** on the inside of the cap.



**To assemble the VixOne properly, squarely align the cap with the cup and press down lightly to fit them together. Rotate slightly to engage the Tabs underneath the bayonet Slides. Then, rotate clockwise to ride down the Slope and across the Flat until slight resistance is felt when the Tab hits the Stop. The cap is now properly installed on the cup. DO NOT OVER-TIGHTEN. DOING SO WILL STRIP THE END STOP AND PREVENT A LEAK-PROOF SEAL.**