

The Next Generation in

PEP Therapy

**Maximum Drug Dosage to the Lungs in Less Time
with Less Waste While Delivering PEP Therapy**

Innovative Therapy
for Airway Clearance




Westmed
When Your Care is Critical

PEP Therapy ORDERING INFORMATION

PRODUCT NUMBER	DESCRIPTION	QTY
0262	Pep Accessory Kit with Manometer	10/cs
0338	Circulaire II with PEP Accessory Kit	10/cs
0290	Aerosol Mask, Adult Non-Vented	50/cs
0291	Aerosol Mask, Pediatric, Non-Vented	50/cs

The Mechanics of PEP Therapy:

Positive pressure is created in the lungs during the expiratory cycle. This is accomplished by reducing the orifice opening of the exhalation resistor. This additional pressure opens the airways and air enters behind the mucus obstruction. Exhalation is used to advance the mucus towards the larger airways, so that the mucus is expelled by the patient.

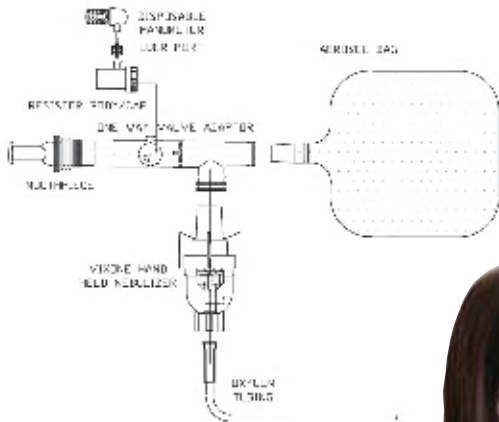

- The combination of PEP therapy & efficient aerosol delivery results in better airway clearance
- Simultaneous therapies save time and reduce costs
 - Simple technique
- Environmental Protection for Healthcare Providers
 - Disposable manometer makes it easy to monitor pressure with minimal obstruction of the patient's face

PEP Therapy

Positive Expiratory Pressure (PEP) therapy drives air into the lungs and behind the mucus using collateral airways. PEP therapy can easily be incorporated when using Circulaire II as it is included as a standard feature. Set the expiratory variable resistor to the minimal opening and pursed lip breathing is simulated as part of the delivery system.

The Circulaire II can be administered with the use of a mask or mouthpiece. By design, the Westmed swivel mask alleviates cumbersome posturing of the patient as this unique characteristic allows you to accommodate your patient's position. These features plus others make for a versatile and universal pulmonary drug delivery system.

The system includes a detachable reservoir bag. Bag replacement accommodates the delivery of a variety of aerosolized medicines such as bronchodilator therapy, antibiotic therapy, mucolytics, antifungals, etc.

U.S. Patent Number 5,020,530 & 5,613,489. Other Patents Pending. © Copyright 2009, Westmed, Inc.
Circulaire is a registered trademark of Westmed, Inc.

Distributed by:

Westmed

When Your Care is Critical

Westmed, Inc.

5580 S. Nogales Hwy.

Tucson, AZ 85706

Phone: 800-975-7987

Fax: 520-294-6061

www.westmedinc.com

